MODE D'EMPLOI - INSTRUCTIONS FOR USE

ASEPTOJET S

ATOMISEUR
ATOMIZER

CE

UGIN
dentaire

Rev 05/16

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IMPORTANT!

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

ASEPTOJET S must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

WARNING

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

CAUTION

▪ **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN'Dentaire**.

▪ **Pictograms:**

General warning: failure to observe the instructions may result in damage to the equipment.

Electrical hazard: failure to observe the instructions may result in electric shock and serious personal injury

INTRODUCTION

The existence and the danger of cross-contamination are now well known.

This situation is worrying, but the problems related to cross-contamination are being addressed by more and more people around the world.

It is important to insist on the fact that, with good conditions, heat, humidity and substratum of growth, only one micro-organism will grow to more than 100 millions in 12 hours.

The efficiency of **ASEPTOJET S** with the disinfecting power of the liquid allow the disinfecting of any space, in a very short time (contact time 45 minutes)

ASEPTOJET S is noiseless, completely automatic, and easy to use. Its time delay, allows the user to leave the room before the appliance is on.

1. TECHNICAL CHARACTERISTICS

- **Maximum volume to be treated:** 450 m³
- **Consumption of liquid:** 1800 ml /hour
- **Volume of tank:** 2 litres
- **Autonomy (2gr/m³):** 1000 m³
- **Dry air compressor, without oil:** 1,6m³ / hour
- **Noise level:** 62 dB
- **Dimensions:**
 - Height: 220 mm
 - Width: 245 mm
 - Length: 390 mm
 - Weight-empty: 14 kg
- **Voltage:** 230V (50Hz / 60Hz)
- **Power:** 300W

2. UNPACK

- Unpack **ASEPTOJET S** and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

3. INSTALLATION AND START-UP

- Open cover **(5)** and place the bottle of disinfecting liquid without cap, in the space reserved to this effect, and then put the cap of the **ASEPTOJET S** on the bottle.
- After checking that the mains voltage matches that noted on the rating plates, plug the cord into a standard 10/16 A socket with a ground terminal.
- Put the appliance in the middle of the room to be treated, and keeping a distance of at least 1 meter from anything (furniture, wall, etc...).
- Turn on the mains switch **(8)** on top of the appliance.
- Adjust the volume knob selector **(6)** in relation to the volume to be treated.
- The user dispose of 30 seconds to leave the room before the start of the appliance.

4. ADVICE FOR USE

- **CURATIVE TREATMENT**

First usage: to provide a maximum decontamination of the room, it is strongly advised, for the first time, to proceed to a complete curative treatment.

For this operation, we advise to set top the timer **(6)** to 450m³ / 30minutes, even if the room is only 100m³ in volume.

- **PREVENTIVE TREATMENT**

Set the timer **(6)** on the estimated volume to be treated (example 50 m³), turn the machine on, it will stop automatically.

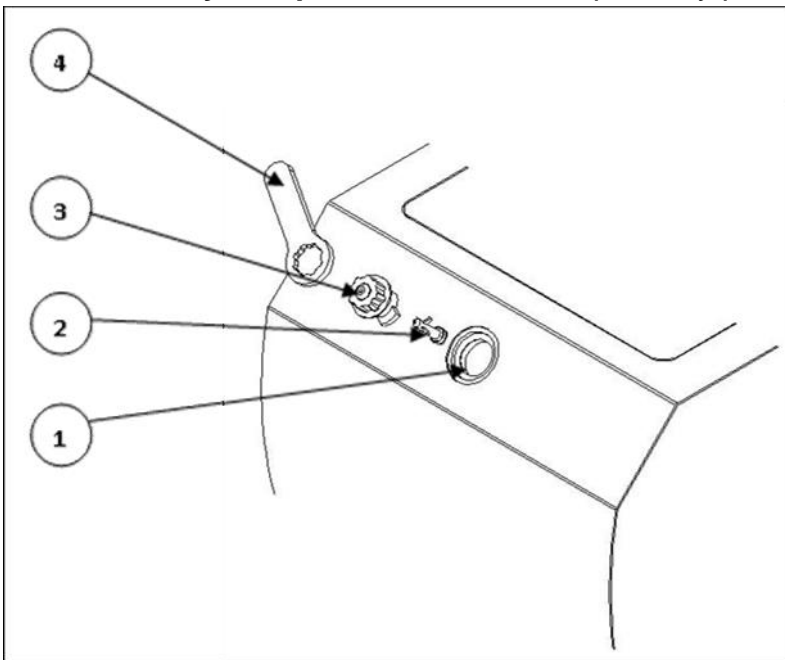
5. MAINTENANCE

ASEPTOJET S must be disconnected from the main power whenever maintenance work and/or cleaning is carried out.

Except the normal cleaning, the atomizer **ASEPTOJET S** does not require any special care.

- A special care must be made about the **nozzle**, which is a very important part of this appliance. The length of the spray must be at least of 1 m to 1,50 m. In case of problems, the key provided **(4)** allows the fast taking apart of this nozzle:
 - Make a ¼ of a turn (counter clock wise).
 - Take off the main grey stud **(3)** of the central part of the nozzle **(2)** and let it soak **(2) + (3)** for 2 hours in chlorinated water.
 - Rinse thoroughly with clean water, and put back the nozzle on the appliance.
- The **air dry compressor** does not require any particular care

- | |
|--|
| 1. Complete nozzle 2. Central part 3. Nozzle insert 4. Nozzle key |
|--|

6. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

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