MODE D'EMPLOI - INSTRUCTIONS FOR USE

# HYDRAUPRESS

PRESSE HYDRAULIQUE  
*HYDRAULIC PRESS*

**UGIN**  
dentaire

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**IMPORTANT!**

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

HYDRAUPRESS mixer must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

**WARNING**

- Do follow carefully all the indications brought back in this manual.
- Don't never try to block the press thrust with and foreign body or with the hands,
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Don't modify the pressure gauge.
- Always verify the level of oil be the mean of special screw cap.

**CAUTION**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN'Dentaire**.

▪ **Pictograms:**

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Information:** carefully read the instructions in the manual

## 1. INTRODUCTION

You have just purchased a hydraulic press and we will thank you. It is particularly appreciated for its ease of use and its robust structure.

## 2. TECHNICAL CHARACTERISTICS

### ▪ Dimensions:

- Height: 470 mm
- Width: 215 mm
- Depth: 190 mm
- Weight: 23 kg

### ▪ Technical data:

- Maximum capacity: 3 flasks + 1 bracket
- Distance between the two plates: 2,5 cm
- Distance between the two columns: 12 cm
- Made entirely of cast iron
- Gauge in glycerine bath

## 3. UNPACK

- Unpack the HYDRAUPRESS and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

## 4. INSTALLATION

- Install the instrument on a plane and stable surface.
- This instrument creates daN / kg 15.000 maximum effort, as consequence it must be handled with high care otherwise it would also be hazardous for the user.

**Don't never try to block the press thrust with and foreign body or with the hands**, it would be very dangerous and there is also the risk to damage the equipment.

## **5. USE**

The HYDRAUPRESS consists of a base containing hydraulic oil. This oil has the function to create an up movement to the piston which, in its turn, create a pressure on the flasks positioned under the hydraulic press. The movement of the piston is indeed enabled by the lever on the mechanic pump.

The maximum piston's stroke is 2,5cm, as consequence it must be adapted to the number of flasks. This distance can be regulated by the mean of the chromated screw; the maximum reachable distance is around 25cm.

Since the operator has fixed the pressure, the instrument maintains always the chosen value.

In order to eliminate the pressure, unscrew the tap placed next to the pressure gauge, which enables a rapid piston release.

The simplicity of the operation of the press hydraulic requires no special maintenance, except the act of observing the tables thrust and respect the powers supported by the flask declared by manufacturers (the pressure must not exceed the red zone).

Give the large force of the hydraulic press it enough to work between 4000 and 7000 daN / Kg thrust (corresponding to the green portion of the gauge – between 100 and 180 bars). Thus, a uniform flow and a constant result that does not damage the gloves.

**Respect the powers supported by the flask** declared by manufacturers (the pressure must not exceed the red zone).

## **6. MAINTENANCE**

As above mentioned the simple structure doesn't require any special maintenance. Use hydraulic oil.

## 7. OPERATIONAL FAULTS

▪ **If the hydraulic press doesn't maintain anymore the same pressure**

1. Check both the O-ring on the mechanic pump and on the piston.
2. Check if the tap/faucet is leaking and if it is well fixed.

Contact after sales services UGIN'Dentaire.

▪ **If the pump doesn't create any pressure**

1. Check if mechanic pump is creating movement inside piston.
2. 2. Check if there is no fouling that clogs the pump.

Contact after sales services UGIN'Dentaire.

## 8. PRESERVATION

The HYDRAUPRESS does not require any particular care or attention in order to be kept in good operating order.

The coated parts guarantee resistance to corrosive agents and the maintenance of the physical and structural characteristics.

If the equipment is not used for long periods, it should be stored in a dry place. If recommend to cover with a suitable cloth to protect it from dust and humidity.

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