MODE D'EMPLOI - INSTRUCTIONS FOR USE

# INTEGRAL 2

POLISSAGE ELECTROLYTIQUE  
*ELECTROLYTIC POLISHING*

CE

**UGIN**  
dentaire

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*Translation into English from original in French*

## D. INSTRUCTIONS FOR USE

### IMPORTANT!

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

**INTEGRAL 2** must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

### WARNING

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

### CAUTION

▪ **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN DENTAIRE**.

### PICTOGRAMS

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Electrical hazard:** failure to observe the instructions may result in electric shock and serious personal injury

## 1. TECHNICAL CHARACTERISTICS

- **Size and weight:**
  - Height: 175 mm
  - Width: 330 mm
  - Depth: 215 mm
  - Weight: 6 Kg
- **Electric power supply:** 230 V – 50/60 Hz
- **Secondary voltage:** 18 V
- **Maximum amperage:** 10 A
- **Timer:** 0 to 30 minutes
- **Circuit-breaker:** 10 A at secondary
- **Fuse :** 2 A at primary
- **Composition vat:** PVC
- **Capacity vat:** 2 litres

## 2. UNPACK

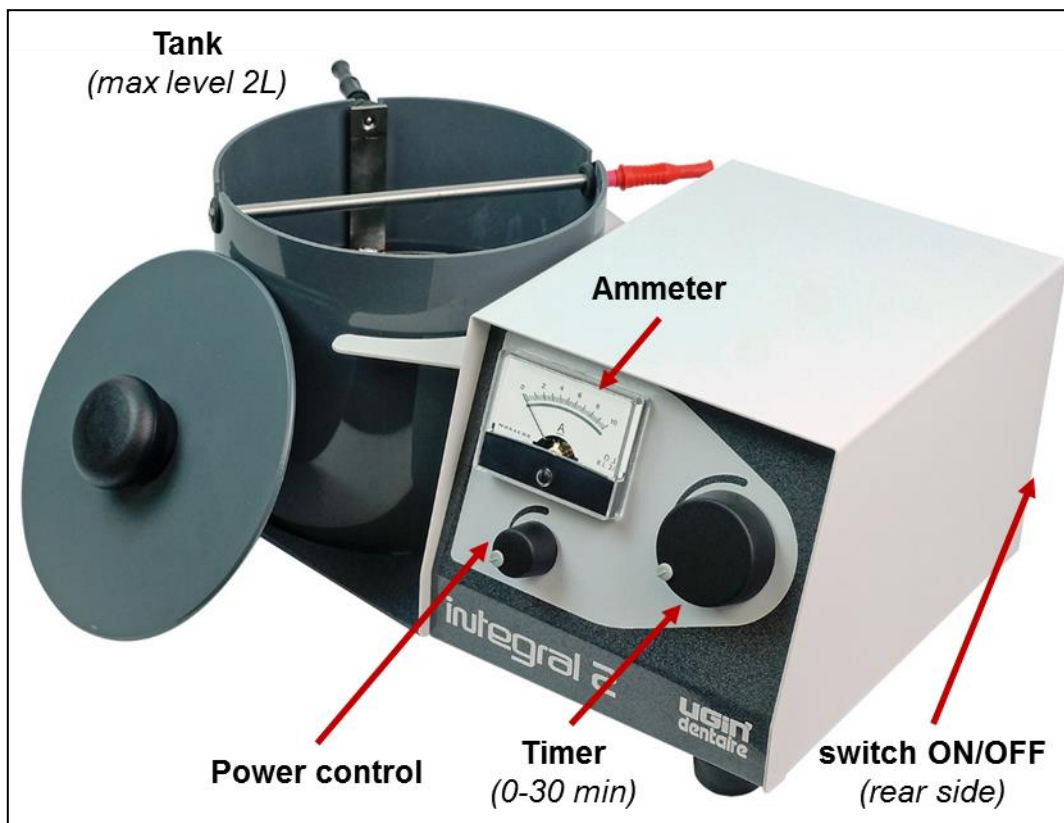
- Unpack **INTEGRAL 2** and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note

## 3. ELECTROLYTIC POLISHING

The **INTEGRAL 2** generator uses the principle of electro-subtraction: the workpiece is submerged in an acid bath, the high current flowing through it allows a quick and effective polishing (5 to 15 minutes).

To achieve a polished surface, electric abrasion decreases height in a process known as electrolytic polishing or electropolishing. This involves electrolysis of the workpiece, attacking the metal, which is then covered by a thin liquid coating with a high metallic ion content. As these ions are less mobile than hydrogen ions, this coating enhances the resistance of the electrolyte to the electric current passing through it. The surface coating varies in thickness, thickest in the hollows and thinnest at the peaks. This makes for a higher current density at the peaks, which thus dissolve more quickly.

## 4. USE

### a) Fill the tank with the polishing liquid

**WARNING:** never exceed the maximum level (max 2L)

We recommend using the **UGIN DENTAIRE** polishing bath: electrolytic liquid - PLS03ALR0001 (1L) and PLS03ALR0002 (3L).

It is exceptionally effective because it heats up rapidly. In addition, it has a particularly good stability over time.

**ATTENTION:** as the polishing quality depends primarily on the kind of solution used. we disclaim results obtained with the solutions other than our own

### b) Connect the workpieces to polishing

A copper wire, supplied with the device, is used to connect the copper bar to the workpiece. Check that proper contact has been made between the two.

- The copper bar should be cleaned regularly with fine-grained emery paper.
- When the copper wire becomes too thin, it should be replaced.

**c) Switch on**

- Connect the power cord to 10/16A grounded power socket.
- Press the power switch ON/OFF (rear side)

**d) Turns the timer (0-30 minutes) to the required time**

To achieve a good polish, the average operating time is 10 to 15 min.

**CAUTION:** do not manually reset the timer to zero (risk of damage to the equipment).  
If necessary, switch off with the power switch ON/OFF.

**e) Adjust the intensity with the power control button**

The current densities most frequently used in electropolishing are around 0,5 A per cm<sup>2</sup>.

The surface area of frameworks may vary considerably, but on an average they are 6 to 10 cm<sup>2</sup>.  
Hence the amperage for a given framework is to 3 to 5 A.

For our machine, we recommend **limiting the total area to be polished to 20 cm<sup>2</sup>, or two to three frameworks, for best results.**

**f) When the polishing time expired, recover the workpieces in the tank**

Use appropriate gloves (observe the recommendations of the manufacturer of polishing liquid - see technical data sheet and/or safety data sheet)

**COMMON PROBLEMS:** special care must be taken with the clasps

Excessive current density or overly long exposure time may dangerously narrow the cross-section of such protruding parts.

At the back of the palate, which is by definition concave, it is normal to observe a lower polish.

A little mechanical polishing may be required to achieve the desired result.

## 5. TROUBLESHOOTING

SHORT-CIRCUIT	POOR CONDUCTION	POLARITY REVERSAL
<p>If the workpiece comes in contact with the annular electrode for several seconds, the thermal circuit-breaker breaks the current. Should this occur, wait for one minute and then reset the circuit-breaker. The device is now ready for operating again.</p>	<p>If the ammeter shows no change, despite the rotation of the potentiometer to the right:</p> <ul style="list-style-type: none"> <li>- Check the connection between the bar and the workpiece,</li> <li>- Check the 10 A circuit-breaker,</li> <li>- Check the fuse.</li> </ul>	<p>If the workpiece builds up a copper layer: Correct the polarity to eliminate the coating.</p>

## 6. MAINTENANCE

**INTEGRAL 2 must be disconnected from the main power** whenever maintenance work and/or cleaning is carried out.

INTEGRAL 2 needs no particular servicing operation.

- **Clean the machine outside** with a soft, damp cloth.  
*ATTENTION: do not use abrasive or solvents for cleaning outside.*
- **Change the polishing liquid periodically.**  
*NOTE: For periodicity, observe the recommendations of the manufacturer of liquid.*
- The **copper bar should be cleaned regularly** with fine-grained emery paper.
- When the **copper wire** becomes too thin, it should be **replaced**.

## 7. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

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