**MODE D'EMPLOI - INSTRUCTIONS FOR USE**

# **MINICAST 2**

---

**FONDEUSE CENTRIFUGEUSE A CHALUMEAU**  
***BLOWTORCH CASTING MACHINE***

CE

**UGIN'**  
dentaire

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**IMPORTANT!**

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

**MINICAST 2** must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

**WARNING**

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

**CAUTION**

▪ **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN Dentaire**.

▪ **Pictograms:**

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Electrical hazard:** failure to observe the instructions may result in electric shock and serious personal injury

## 1. TECHNICAL CHARACTERISTICS

### ▪ Dimensions:

- Height: Cover closed: 330 mm; Cover open: 740mm
- Width: 520 mm
- Length: 710 mm
- Net weight: 50 kg

### ▪ Power requirement: single phase 230v

### ▪ Frequency range: 50Hz or 60HZ

### ▪ Power consumption: 300W

### ▪ Arm rotation speed: 500rpm

### ▪ Crucible capacity: 60gr of alloy

### ▪ Injection time-delay: 6 to 60 seconds

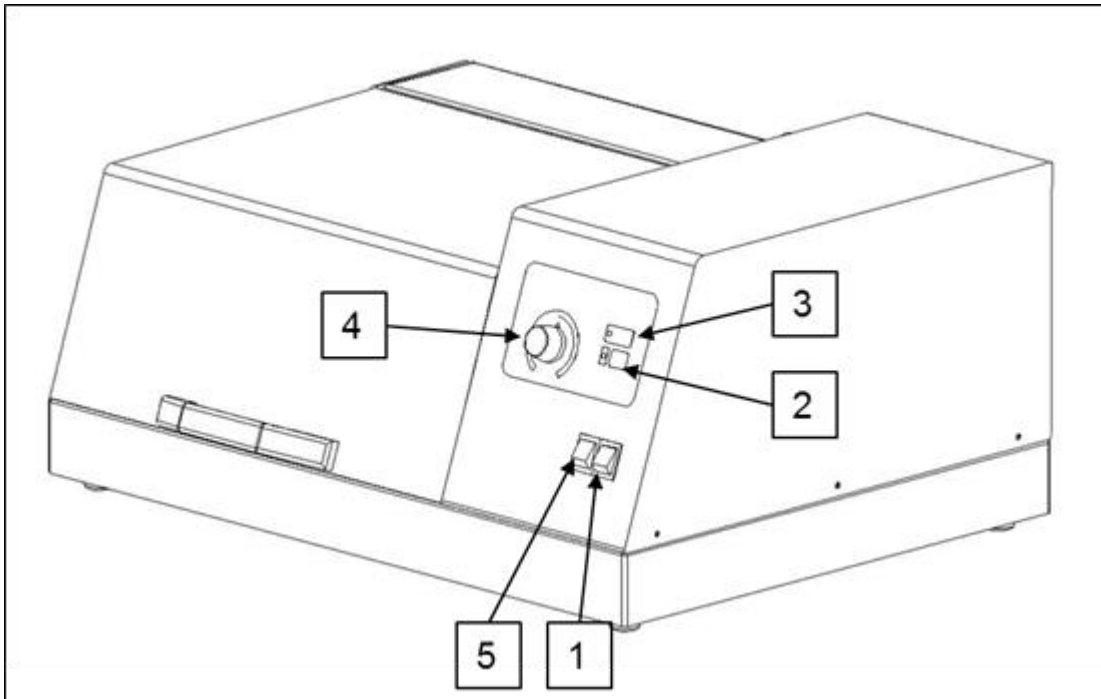
## 2. UNPACK

- Unpack **MINICAST 2** and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

## 3. INSTALLATION AND START-UP

- Set the machine on a flat surface and level it if needed.
- Raise the cover and remove the arm block.
- After making sure that the mains voltage corresponds to that mentioned on the identification plate, connect the power cord to a standard 10/16A plug with a grounding terminal.

#### 4. DESCRIPTION OF THE CONTROL PANEL

<i>Control panel</i>	
1	Power On / Off key
2	Cycle start key
3	Door safety lock indicator
4	Injection time-delay (6 to 60 secondes)
5	Selector of arm acceleration (slow / fast)

##### **1) Power On/Off key**

This key switches indicate the power on. It may also be used in the event of a problem for **emergency**.

##### **2) Cycle start key**

Before beginning a melting operation, you must make sure to press this key (with the cover open).

The indicator coupled with the key lights to confirm that the cycle start command has been registered.

When you close the cover, the arm will automatically begin rotating.

Advantage: The user, with the torch in one hand and the cover handle in the other, does not need to enter any commands on the control panel during melting and injection.

**3) Door safety lock indicator**

During arm rotation, the cover is locked by an electromagnetic latch.  
The DOOR SAFETY LOCK indicator lights to confirm that the cover is locked.

**4) Injection time delay (6 to 60 seconds)**

The potentiometer is used to set centrifuging time. The count down begins as soon as the cover is closed.

The arm stops automatically at the end of the hold time.

- Waiting time for non-precious metals : 10 to 15 seconds
- Waiting time for precious metals: 20 to 25 seconds

**5) Selector of arm acceleration (slow / fast)**

This switch with two positions allows a choice of speed for arm rotation.

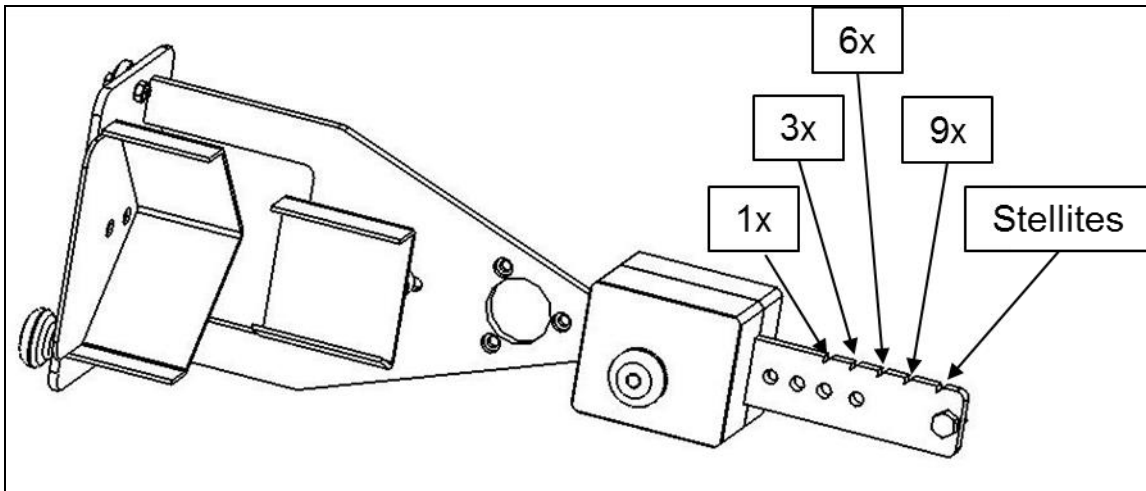
**IMPORTANT: DO NOT CHANGE THE POSITION OF THE SELECTOR DURING THE CYCLE**

## 5. MELTING AND INJECTION

After installing your machine according to § 3 – *Installation and star up*:

- Press the ON/OFF key
- Set the time-delay potentiometer (roughly 1/4 turn above the minimum)
- Open the cover
- Position the casting ring and adjust its height using the lever
- Position the counterweight according to the size of the casting ring

- Put the metal in the crucible
- Position the crucible against the casting ring
- Press the CYCLE key
- Melt the alloy with the torch
- When the alloy is ready to be injected, close down the cover which will make the motor starts

**NOTE: The rotation way is counter clockwise**

## 6. MAINTENANCE

**MINICAST 2 must be disconnected from the main power** whenever maintenance work and/or cleaning is carried out.

Except the normal cleaning, **MINICAST 2** does not require any special care.

## 7. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

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