MODE D'EMPLOI - INSTRUCTIONS FOR USE

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## *OIL-LESS VACUUM PUMP*

CE

**UGIN**  
dentaire

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*Translation into English from original in French.*

## D. INSTRUCTIONS FOR USE

### 1. SAFETY INSTRUCTIONS

#### IMPORTANT!

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

This oil-less vacuum pump must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

#### WARNING

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Use only the accessories in equipment for the filling operations and/or maintenance.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

#### CAUTION

- **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN'Dentaire**.

#### *Pictograms:*

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Electrical hazard:** failure to observe the instructions may result in electric shock and serious personal injury

**Hot surface:** risk of burns

**Information:** carefully read the instructions in the manual

## 2. TECHNICAL CHARACTERISTICS

- **Dimensions:**
  - Height: 210 mm
  - Width: 155 mm
  - Depth: 250 mm
  - Weight: 7.6 kg
- **Electrical supply:** 230V (50/60Hz)
- **Flow:** 54 L/min (50Hz) – 63 L/min (60 Hz)
- **Vacuum limit:** 745 mmHg - 29,33 inHg – 993.25 mbar

## 3. UNPACK

- Unpack the pump and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

## 4. INSTALLATION

### ○ **Intended location**

- For correct operation, the pump must be used in a well-ventilated room, without humidity, and with low levels of dust. The room temperature should not exceed 40° C.
- If it is not used, keep it in a room without humidity and protect it from dust and vibrations.
- It is recommended that the device be levelled horizontally.
- Keep around ventilation enough space to ensure good ventilation (at least 150 mm). Do not obstruct the ventilation system for protection against overheating

**ATTENTION: do not place the apparatus inside a piece of furniture without circulation of air.**

### ○ **Gas usage**

- This vacuum pump was designed to pumping non-aggressive gases containing oxygen less than 22%. These gases must not be toxic or explosive.
- Never use this pump to suck up corrosive liquids or water vapour.
- The permissible maximum gas temperature at the inlet is 40 ° C

## 5. COMMISSIONING

The vacuum pump has standard accessories of one silencer and one set of rubber feet 4 pcs):

- The silencer is for the purpose of lowering noise (upper part of the device)
- the rubber feet is for reducing vibration during operation.

### COMMISSIONING

1. Connect the “Tricolair” tube to the ribbed elbow connector.
2. Connect the other end of this tube to the kiln (at rear).  
**NOTE: respect the direction of connection**
3. Connect the power supply cable to the pump from the kiln socket (at rear).

**Warning: risk of burns!**

The surface temperature of machine parts may exceed 60°C.  
Take steps to prevent people from accidentally touching the hot parts.

## 6. MAINTENANCE

**The vacuum pump must be disconnected from the main power** whenever maintenance work and/or cleaning is carried out.

- This device does not require any special maintenance.
- No need to lubricate.
- Never clean the pump with corrosive products or acids.
- All maintenance operations must be performed by qualified personnel.

### CLEANING

- **Regularly**
  - Clean the external surface of the machine in order to remove any dust.
  - Clean the whole ventilation section.
  - Never clean the pump with corrosive products or acids.
- **Annually**
  - Dismantle the silencer and clean it. Change it if necessary.
  - Replace the suction filter cartridge

## 7. TROUBLESHOOTING

<i>Problems</i>	<i>Possible cause</i>	<i>Solution</i>
<b>Motor doesn't start</b>	a) No power source	a) Check main leads
	b) Insufficient voltage	b) Check the voltage
	c) Malfunction of capacitor	c) Check capacitor, if necessary, replace it
	d) Wrong circuit connection	d) Check the electrical connection
	e) Motor is out of order	e) Send back for maintenance
	f) Piston jammed	f) Dismantle the cylinder head and check. If necessary, replace the piston rod
<b>Motor overheated</b>	a) Ambient temperature overheat	a) Choose a more appropriate location and provide better ventilation
	b) Poor ventilation	b) Clean the fan(s) grid and check that the motor fan(s) is (are) not too close to a wall or partition
<b>Insufficient vacuum or pressure</b>	a) Leak in installation	a) Check and block the leaks
	b) Valve flapper broken	b) Replace it
<b>Abnormal noise</b>	a) Valve flapper damaged	a) Replace it
	b) Not suitable silencer	b) Choose suitable silencer and replace it
	c) Severe vibration	c) Replace cushion rubber feet

## 8. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

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