## MODE D'EMPLOI - INSTRUCTIONS FOR USE

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*Translation into English from original in French*

## E. INSTRUCTIONS FOR USE

### 1. SAFETY INSTRUCTIONS

#### 1.1. USE

Steam cleaner PRESSOJET + to manual fill has to be operated only by dental professionals.

PRESSOJET + has to use in the conditions and for the uses described in this manual. Any other use that will be made of it is considered inappropriate. In this case, the manufacturer declines all liability and immediately cancels the equipment warranty.

According to the product's instructions and functionalities, **it is imperative to read carefully the instructions for use before installation and use.** Retain the manual for future reference.

#### 1.2. SAFETY INSTRUCTIONS

Pictograms' meaning used in these instructions for use:

	General warning		Electrical hazard		Burn hazard - Hot surface
---	-----------------	---	-------------------	---	---------------------------

Observe following safety instructions:

	<ul style="list-style-type: none"> <li>- During operation and after use, <b>the surface and steam gun may reach high temperatures: do not touch</b></li> </ul>
	<ul style="list-style-type: none"> <li>- The equipment has to be connected to an <b>electrical installation in accordance</b> with the applicable standard in the country in which it is being used. This installation has to provide <b>protection against overcurrent, overload and ground faults</b></li> <li>- <b>Connect it directly to the mains socket.</b> Do not use multiple sockets</li> <li>- Ensure that the <b>connection cables</b> are in perfect condition to prevent short circuits</li> <li>- Before any <b>technical intervention</b>, the equipment has to be switched off (with the breaker) and disconnected from the mains power supply</li> <li>- <b>Do not spill liquid</b> on the ventilation openings or inside the equipment</li> </ul>
	<ul style="list-style-type: none"> <li>- The equipment must be placed on a <b>flat surface</b>, keeping sufficient clearance around the unit to <b>ensure proper ventilation</b></li> <li>- <b>Do not block the ventilation openings</b> to avoid overheating</li> </ul>
	<ul style="list-style-type: none"> <li>- Use only <b>genuine spare parts</b>. The use of non-original spare parts voids any warranty for your device</li> <li>- <b>Do not introduce stranger objects inside</b> the equipment during the maintenance operations. It is forbidden to modify the material without authorization</li> <li>- <b>Do not execute different operations of maintenance</b> from those brought back in the manual. Whichever operation not included in this manual, can involve risks</li> <li>- For whichever information concerning installation, maintenance and use, contact the <b>customer's service UGIN DENTAIRE</b></li> </ul>

### 1.3. WASTE DISPOSAL

Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.

The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.

## 2. EC DECLARATION OF CONFORMITY

The equipment’s EC declaration of conformity is provided as appendix of instructions for use.

### Content of the declaration EC:

- Name and address of the manufacturer:  
UGIN DENTAIRE. 25 rue de la Tuilerie. 38170 SEYSSINET-PARISSET – FRANCE
- UGIN DENTAIRE declares, on its own exclusive responsibility, that the product:
  - Steam cleaner to manual fill PRESSOJET +
  - Serial number: *see the equipment’s EC declaration of conformity*
- Fulfills the essential requirements of health and safety of the following directives:
  - Machinery directive 2006/42/EC
  - Electromagnetic compatibility directive 2014/30/EU

## 3. TECHNICAL INFORMATION

### 3.1. TECHNICAL DATA

Steam cleaner PRESSOJET + to manual fill has to be operated only by dental professionals and in the conditions described in this manual (*see chapter 1.1 and 3.2*).

<b>TECHNICAL DATA</b>	
<b>Dimensions</b> <i>(height x width x depth)</i>	320 x 300 x 320 mm
<b>Weight</b>	12 kg
<b>Voltage</b>	220/230 V – 50/60 Hz
<b>Power consumption</b>	1 600 W
<b>Stainless steel tank capacity</b>	4,5 litres
<b>Operating pressure</b>	5 bar
<b>Temperature rising time</b>	15 – 20 minutes

### 3.2. CONDITIONS OF USE, TRANSPORT AND STORAGE

#### ▪ **Authorised conditions of use**

- Ambient temperature: +5°C to +40°C
- Humidity range: maximum relative humidity 80% for temperatures until 31°C, without condensation and decrease linear until 50% to 40°C, without condensation
- Ambient pressure: up to an altitude of 2000 m above sea level

#### ▪ **Authorised conditions of storage**

- Ambient temperature: -20°C to +65°C
- Humidity range: maximum relative humidity 80%
- Ambient pressure: up to an altitude of 2000 m above sea level

#### ▪ **Conditions to transport**

- Wait the complete cooling before packing/transporting
- Use the original packaging only, with the protective elements
- Install the equipment in the packaging with the protective elements.

## 4. INSTALLATION AND START-UP

### 4.1. UNPACK

- Unpack PRESSOJET + and check that it is in perfect order
- Any defects can be reported to the carrier
- Remember to get the carrier to sign the delivery note

*NOTE: keep the original packaging to transport the equipment (see chapter 3.2)*

### 4.2. INSTALLATION

- Install in a room aired. Observe the conditions indicated in chapter 3.2
- Place the equipment on a level and aired surface, easily accessible. Keep a distance from other objects around not less than 10 cm.
- Keep it away from heat sources (for example, radiators and/or other equipment that release heat). Avoid vibrations and shocks
- Do not install in a place with explosion hazard
- Its forbidden placing inflammables, toxic, volatile or explosives around the equipment

**Grounding is mandatory** to avoid accidents and the dangers of overheating the power line.

Due to the power of equipment: **connect it directly to the mains**, do not use multiple sockets (risk of electrical interference)

### 4.3. START-UP

*NOTE: for marks, see chapter A – equipment description*

- Remove screw cap
- Fill the tank with 4,5 litres of distilled water (or demineralised water), using the special container provided
- Replace screw cap after PRESSOJET + has been filled

**IMPORTANT: never exceed the maximum level (max 4,5L)**

**To ensure long-term performance of the equipment, it is essential that only distilled (or demineralised) water be used. Do not use any other substances**

- Plug in power cord to a properly grounded 230 V electrical receptacle
- Once the tank filled with 4,5 litres of distilled (or demineralised) water and closed cap securely, turn on power switch (*START*)
- The heating indicator red will be illuminated immediately
- Wait for the heating indicator turned off (approx. 15-20 minutes) to start working (when the operating pressure is reached). The needle of the pressure indicator moves into the blue area

**ATTENTION: Never try to unscrew the cap when the unit is switched on**

- Adjust the desired steam flow with the steam adjust knob located on the steam handgun
- Press button on the steam handgun to release steam
- The green light comes on when the steam gun is activated
- During the use, point the steam handgun only the job direction area. Do not point in another direction
- Turn off the machine (*START*) when you have finished using it and place the handgun

**Always place the handgun in the holder after use**

#### 4.3.1. REFILL THE TANK

When the water level is low, proceed to filling the tank respecting the following instructions:

1. turn off the machine
2. wait for the pressure indicator to return to zero (0)
3. carefully unscrew filler cap (to release any remaining hot steam)  
*Caution: hot steam – burn hazard*
4. wait a few minutes longer before starting to filling
5. fill the tank with 4,5 litres of distilled water (or demineralised water), using the special container provided

**HOT SURFACES** : during operation, the cover and the gun steam outlet may become very hot: avoid direct contact with the skin (burn hazard)

## 5. MAINTENANCE

**Before to carry out the maintenance work and/or cleaning:**

- **disconnect from the main power supply** (*switch OFF and unplug to mains socket*)
- **wait the cooling at room temperature** after use (*burn hazard and/or damage*)

Observe the following recommendations to ensure the longevity of the device:

- Clean the machine outside with a soft, dry cloth
- Do not use abrasive, solvents or flammables products for cleaning (outside and inside)
- **Replace the gasket cap every three months**  
*Remove old gasket located on underside of the screw cap and replace with a new gasket*
- **To conduct a regular descaling of the tank** to ensure long-term performance to this device  
*Recommended periodicity: every three months or more often for intensive use*

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