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Translation into English from original in French

F. INSTRUCTIONS FOR USE

1. SAFETY INSTRUCTIONS

1.1. USE

Sandblaster UGISAB has to be operated only by dental professionals.

UGISAB has to use in the conditions and for the uses described in this manual. Any other use that will be made of it is considered inappropriate. In this case, the manufacturer declines all liability and immediately cancels the equipment warranty.

According to the product's instructions and functionalities, **it is imperative to carefully read the instructions for use before installation and use**. Retain the manual for future reference.

1.2. SAFETY INSTRUCTIONS

Pictograms' meaning used in these instructions for use:

	General warning		Electrical hazard
---	-----------------	---	-------------------

Observe following safety instructions:

	<ul style="list-style-type: none"> - The equipment has to be connected to an electrical installation in accordance with the applicable standard in the country in which it is being used. This installation has to provide protection against overcurrent, overload and ground faults. - Connect it directly to the mains socket. Do not use multiple sockets. - Ensure that the connection cables are in perfect condition to prevent short circuits. - Ensure that the pneumatic hoses (power socket) are in perfect condition. - Before any technical intervention, the equipment has to be switched off (with the breaker) and disconnected from the mains power supply. - Do not spill liquid on the ventilation openings or inside the equipment.
	<ul style="list-style-type: none"> - Use only genuine spare parts. The use of non-original spare parts voids any warranty for your device. - Do not introduce stranger objects inside the equipment during the maintenance operations. It is forbidden to modify the material without authorization. - Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. - For whichever information concerning installation, maintenance and use, contact the customer's service UGIN DENTAIRE.

1.3. WASTE DISPOSAL

Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.

The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and has to be disposed of in accordance with applicable regulations.

2. EC DECLARATION OF CONFORMITY

The equipment's EC declaration of conformity is provided as appendix of instructions for use.

Content of the declaration EC:

- Name and address of the manufacturer:
UGIN DENTAIRE. 25 rue de la Tuilerie. 38170 SEYSSINET-PARISSET – FRANCE
- UGIN DENTAIRE declares, on its own exclusive responsibility, that the product:
 - **Sandblaster UGISAB**
 - Serial number: *see the equipment's EC declaration of conformity*
- Fulfills the essential requirements of health and safety of the following directives:
 - Machinery directive 2006/42/EC
 - Electromagnetic compatibility directive 2014/30/EU

3. TECHNICAL INFORMATION

Sandblaster UGISAB has to be operated only by dental professionals and in the conditions described in this manual (*see chapter 1.1*).

TECHNICAL DATA	
Working pressure	Minimum 2 bar – Maximum 6 bar
Air consumption	100 L/min to 4 bar
Blasting nozzle	Ø 3.0 mm in tungsten carbide
Lighting	Compact energy – 42 LED
Dust filtration	Connection to the WAFIS or PRO-3 SHAKE filtering system (<i>in option, not included</i>)
Voltage	230 V – 50/60 Hz
Power	6 W – 0.2 A
Dimensions	
- Height	390 mm
- Width	350 mm
- Depth	290 mm
Net and gross weight	8.6 kg and 115 kg

4. INSTALLATION AND START-UP

4.1. UNPACK

- Unpack UGISAB and check that it is in perfect order.
- Any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

NOTE: keep the original packaging to transport the equipment.

4.2. INSTALLATION

NOTE: for marks, see chapter B – Spare parts.

- Place the machine on the work bench, on a level surface.
- The glass window (No. 14) still has to be assembled to the machine since packed separately to avoid any damages during transport.
For the assembly, place the window (No. 14) on the rubber joint (No. 45), correctly positioned on the hinges (No. 47) and force the metallic pins (No. 7) to be fitted in (it is recommended to introduce the pins with the knurled part towards the inside of the machine).
- Make sure that the mains voltage complies with the voltage indicated on the rating plate, connect the power cord (No. 37) to a 10/16 A grounded power socket.

Grounding is mandatory to avoid accidents and the dangers of overheating the power line.
Connect it directly to the mains, do not use multiple sockets.

- Check that the hoses and connectors are in good condition.
- Assemble on the pressure reducer (No. 42).
- Perform the pneumatic connection to the compressed air system of the laboratory:
 - if in Rilsan or polyethylene 8x6, screw the quick clutch (No. 40),
 - if in rubber 12x6, screw the air connection (No. 41) and tighten by means of a hose clamp.
- Connect the pedal control on the pipefitting (No. 23) following the different colours.
- Fill the working chamber with the appropriate abrasive according to the operations to be carried out (approx. 6 Kg) as specified in the above table.

TABLE OF ABRASIVE

NAME	CODE	DESCRIPTION
CRISTALBA	SAB03UGI0004 (10kg)	Corundum white grain 46 (350µ) for non-precious alloys (may be used with standard nozzle Ø 3,0) - pressure 4/5 bar
CRISTALBA	SAB03UGI0005 (10kg)	Corundum white grain 60 (250µ) for precious alloys (may be used with standard nozzle Ø 3,0) - pressure 3/4 bar
ALUMINE	SAB03UGI0011 (10 kg)	Corundum white grain 60 (250µ) for precious alloys (may be used with standard nozzle Ø 3,0) - pressure 3/4 bar
UGIN'BLAST	SAB03UGI0008 (3 kg) SAB03UGI0009 (25 kg)	Corundum white grain (200µ) for the satin-finishing of any metal (may be used with standard nozzle Ø 3,0) - pressure 3/4 bar

4.3. CONNECTING THE SYSTEM VACUUM

ATTENTION: the sandblasting process originates harmful dust: do not breathe. It is imperative to connect a dust extraction system (WAFIS or PRO-3 SHAKE)

- To connect the **WAFIS or PRO-3 SHAKE** dust filtering system, it is sufficient to introduce the suction pipe on the flange (No. 10). It is also possible to fit a pipe with internal Ø 30 or 40 mm.
- Plug in the electric feeding of the extractor to the socket (No. 21), assuring the automatic operation.

4.4. START-UP

- Switch on light switch (No. 38), if the dust extraction system connected to the machine is automatically activated by the machine.
- Introduce the part to be blasted inside the working chamber and close the window locking pin (No. 19 – assuring the perfect airtight seal on the joint). We remind to close hermetically the machine every time you start to work.
- Lift the reducer knob (No. 42) and turn it till the desired pressure is set, push the knob down to block.
- To start blasting, operate the foot control (No. 22).

NEVER OPEN THE GLASS WINDOW BEFORE THE COMPRESSED AIR FEEDING VALVE HAS BEEN DULY CLOSED

5. MAINTENANCE AND TROUBLE-SHOOTING

Before to carry out the maintenance work and/or cleaning, disconnect from the main power supply (switch OFF and unplug to mains socket) and pneumatic system.

Observe the following recommendations to ensure the longevity of the device:

- Clean the machine outside with a soft, dry cloth.
- Do not use abrasive, solvents or flammables products for cleaning (outside and inside).

5.1. REPLACE

Description of the procedure to be followed to replace several components of UGISAB:

COMPONENT	PROCEDURE TO BE FOLLOWED
Lamp (No. 5)	Remove the plastic cover (No. 6) from its support (No. 4), take off the lamp rightward and re-assemble.
Worn out nozzle (No. 29)	Remove the nozzle-holder cap from the body (No. 28) and introduce a new unit in the same position.
Plastic protection screens of the glass window (No. 17)	Remove the fixing clips (No. 16), clean accurately the glass and place a new protection screen.
Gloves (No. 18)	Unscrew the four screws to remove the flange (No. 15), put the new glove on the flange holding band and re-assemble.
Foot-control air feeding valve (No. 22)	By means of a small screwdriver, loosen the pliers nut, keep the spring and the sphere, disconnect the pipes, unscrew the fixing screw and replace.
Worn out abrasive	Move the chamber forward, remove the cap (No. 35) from the lower part of the machine and let abrasive flow through the exhaust hole. Clean accurately and fill in the hopper with new abrasive (approx. 6 kg).

5.2. TROUBLE-SHOOTING

PROBLEME	CAUSE	PROCEDURE TO BE FOLLOWED
No lighting	1. Bad electrical connection	Check that the plug is correctly plugged in.
	2. Lamp is burnt	Replace the lamp (<i>see chapter 5.1</i>).
	3. Switch is damaged	Replace the switch.
No air	1. Bad pneumatic connection	Check connection to the compressor.
	2. Reducer is completely down	Verify pressure parameters and setting.
	3. Foot-control valve faulty or dirty	Check, clean or replace.
	4. Internal pipes are clogged	Check the connections and condition of pipes up to the outlet end (nozzle).
Sandblasting irregularities	1. Compressor capacity is not sufficient	Check compressor features: the minimum capacity must be of at least 150 litres per minute (to assure steady power) and a tank of 100 litres (to assure autonomy).
	2. Pressure is not suitable to the metal to be treated	Follow instructions of the abrasive table (<i>chapter 4.2 - Installation</i>)
	3. Abrasive is not suitable	Follow instructions of the abrasive table (<i>chapter 4.2 - Installation</i>)
	4. Abrasive is worn out	Replace.
	5. Tungsten nozzle is not suited to the abrasive grain size or it is clogged	Follow instructions of the abrasive table (<i>chapter 4.2 - Installation</i>). Replace.
Spraying nozzle dispenses only air, no abrasive	1. Tungsten nozzle is worn out	Replace.
	2. Wrong position of nozzle – no abrasive suction	The nozzle is correctly positioned only when it comes out from the nozzle-holder.
	3. Either the nozzle or the pipes are logged	Check if some investment particles have entered the suction pipe or the nozzle body.
No air comes out from the projector	1. The projector inside nozzle is clogged	Remove the cap and clean the injector.
	2. Blasting valve is clogged or dirty	Close the air intake, disassemble the valve and check.
Nozzle-holder cap comes out during sandblasting	Pipes and nozzles are obstructed or the abrasive is too rich in investment slags	Stop up the end of the nozzle and operate the valve; air flow will thus be inverted eliminating any residuals from the piping. Replace the abrasive.
Internal loss of air	Leaks at pipe connections	Check the waterproofness of connectors.
Dust is leaking out from the glass window	1. Joint is not perfectly airtight	Check plan of the sheet body, condition of the joint and pressure of the closing dowel on the window.
	2. The filter is clogged or the extraction system is not sufficient	Check regularly condition of the dry filter or of the extraction system.

If the problem persists, contact the after-sale service.

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