MODE D'EMPLOI - INSTRUCTIONS FOR USE

# UGIVAC 5

MALAXEUR SOUS VIDE  
*VACUUM MIXER*

CE

**UGIN'**  
dentaire

Rev 04/16

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**IMPORTANT!**

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

UGIVAC 5 mixer must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

**WARNING**

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet, and/or disconnect the compressed air.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.
- Never use your fingers or a utensil to stop the micro-motor as this may cause injury and seriously damage the mixer.

**CAUTION**

- **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN'Dentaire**.

- **Pictograms:**

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Electrical hazard:** failure to observe the instructions may result in electric shock and serious personal injury

**Information:** carefully read the instructions in the manual

## 1. INTRODUCTION

This equipment is easy to operate for vacuum mixing investments, plasters and silicones in your dental prosthesis laboratory.

## 2. TECHNICAL CHARACTERISTICS

- **Dimensions:**
  - Height: 180 mm
  - Width: 255 mm
  - Depth: 230 mm
  - Weight: 11 kg
- **Voltage:** 230V (50Hz / 60Hz)
- **Power:** 100W

## 3. UNPACK

- Unpack the UGIVAC 5 and check that it is in perfect order, so that any defects can be reported to the carrier.
- Verify integrity of the pack at the moment of the reception of the apparatus.
- Remember to get the carrier to sign the delivery note.
- Verify that in the packaging all the parts and the supplied accessories of series are contained together with the apparatus:
  - 1 mixing bowl (250cc and 500 cc) c/w lid and paddle **(9)**
  - 4 screws and plugs
  - 1 6x8 extraflex hose. Length: 2 metres (for connecting to laboratory compressed-air supply).
  - User's documentation
  
  - OPTIONAL: 1 stand (order separately).

## 4. INSTALLATION

- Undo the M5 screw at the bottom of the UGIVAC 5 mixer and remove the wall mounting unit.
- Place the mixer wall bracket in position on the wall. Next, drill the holes and insert the screw plugs. Secure the angle bracket to the wall using the 4 screws provided.

### **IMPORTANT!**

Use suitable screws and plugs when attaching the mixer to thin walls (e.g. plasterboard)

- Place the UGIVAC 5 mixer on the wall bracket and slot into position using the 2 dowels at the rear.
- Screw the mixer onto the angle bracket with the M5 screw.
- Connect the 6x8 extraflex hose to the quick-release coupling **(2)** (compressed-air supply).
- Make sure that the mains voltage matches the voltage shown on the rating plate; next, plug the power lead into a standard 10/16 A earthed socket.

## 5. HOW TO OPERATE THE MIXER:

- The supply pressure to the mixer should be adjusted when it is used for the first time:
  - Use the control dial **(1)** to set the pressure.
  - The pressure gauge **(3)** displays the pressure.
  - for best mixing results, it is advisable to select a pressure between 3-4 bar in order to get a level of vacuum corresponding 50-60 mmHg (this value is visible on Vacuum pressure gauge **(5)**)
- Press the green "POWER" button **(8)** to start creating the vacuum.

- Place the mixing bowl, 250cc or 500cc **(9)**, in the centre of the mixer base in the bay provided. The lid contains a drive nut that rotates the paddle. Turn the bowl to the right or left if necessary so that it is firmly attached to the mixer.

The mixing bowl will be held in place by the vacuum pressure that is created.

***NB:*** the "Venturi" used to create the vacuum will sound as though it is discharging air: this is entirely normal and does not indicate a malfunction.

- The duration of the mixing cycle can be set between 0'00" and 9'59" using the +/- keys **(7)**.
- Press the "MIXING" button **(4)** to begin the mixing cycle.

**Never exceed the maximum level indicated on the bowl:** a product of excess leads to clogging of the suction system and may seriously damage the mixer.

- The mixing countdown display **(6)** shows the time remaining until the end of the cycle (0'00").
- Press the "Power" button **(8)** again to end the vacuum and release the mixing bowl.

**Never use your fingers or a utensil to stop the micro-motor as this may cause injury and seriously damage the mixer**

## 6. MAINTENANCE

**UGIVAC 5 must be disconnected from the main power** whenever maintenance work and/or cleaning is carried out. The electrical connection must be performed by a qualified electrician.

Apart from cleaning the outer surfaces, this equipment requires little maintenance:

- Check the pneumatic couplings from time to time.
- Keep those areas that hold the bowl in line with the mixer clean so as to maintain sufficient vacuum pressure.
- Clean the bowls and paddles thoroughly after use.

## 7. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

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